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Tel +39 0185 939481 - Fax +39 0185 938595

info@larident.it - www.larident.it

MANUFACTURER'S INSTRUCTIONS FOR THE TREATMENT OF MEDICAL DEVICES ACCORDING TO THE EN ISO 17664:2017 STANDARDS

MANUFACTURER: Larident Srl

REF: C90/C92/C99/C95

WARNING NOTICE: THE PRODUCT IS NOT SUPPLIED STERILE.
STERILIZE IT BEFORE USE

Restriction on treatments: the product is for single use

Area of use: support and cleaning of endodontic instruments

Storage and transport: the product does not have specific needs

Method of use: clean the file or reamer files by rotating the instrument, guided by a hand inside the sponge. The impurities left in the endodontic instruments will be removed by the porosity of the sponges

Disinfection and sterilization: sterilize in an autoclave machine before use at 134° for 4 minutes. The product could slightly change colour but this will not change its usage or performance

Maintenance: no specific requirement

Disposal: dispose of the product according to the correct disposal procedures

Function check and test: visual check for any damage or deformation of the product

Storage: the product does not have specific needs

The instructions listed above have been validated by the manufacturer of the medical devices as suitable for preparing the device for use.

It is the user's responsibility to ensure that the preparation process, the equipment the materials used and the staff obtain the desired results.

Any deviation from the recommended instructions must be carefully checked with regards to its efficiency and possible adverse consequences.

LARIDENT S.r.l.

Via Lamaineigra 12R (Ponte Rimassa)

16030 Tribogna (Genova) - ITALY

Tel +39 0185 939481 - Fax +39 0185 938595

info@larident.it - www.larident.it